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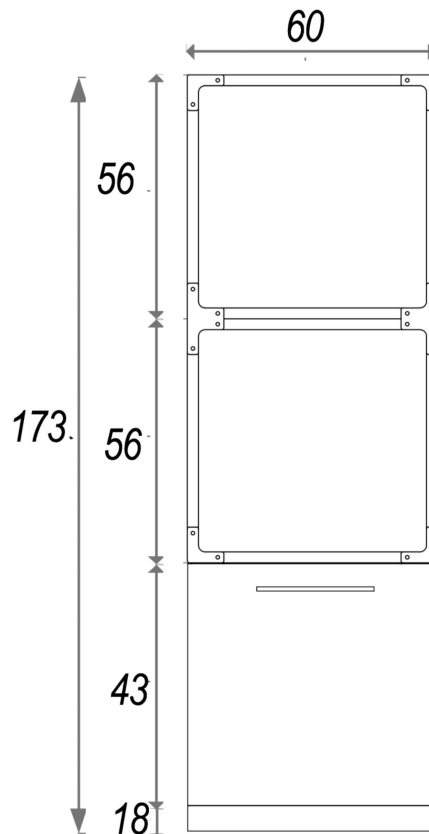
## RECOVERY CAGE

### GA042

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#### STANDARD RECOVERY CAGE 2 COMPARTMENTS ON CABINET

|              |                          |
|--------------|--------------------------|
| MATERIAL     | STAINLESS STEEL AISI 304 |
| COMPARTMENTS | 2                        |
| PARTITIONS   | 0                        |
| DRAINS       | NO                       |
| WHEELS       | NO                       |

Paneled on the left and right side.  
Cabinet with one door with shelf arranged on two floors.  
External dimensions: L60xP72xH173 cm

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The undersigned Lory Progetti Veterinarians srl, established in Via B. Buozzi 3/A, Cavriago, RE DECLARE That the equipment of which follows the characteristics: Denomination: GA042  
The appliance is built with materials corresponding to the European regulations for medical use in the veterinary field. The instrument does not contain any electrical parts and has a manual operation. Complies with the following directives: 2001/95/CE (general product safety) 2004/108/EC (Electromagnetic compatibility).

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### TREATMENT INSTRUCTIONS FOR AISI 304 STAINLESS STEEL

#### FIRST CLEANING

Before using equipment made of aisi 304 stainless steel, it's recommended to clean the entire internal and external surface to remove any possible working residual; according to the type of equipment,

#### FOLLOW THIS INDICATIONS

- On a surface that will come into direct contact with water you must use a no-aggressive or anti-calcare product. (Es like vialac)
- On a surface that will not come everytime into direct contact with water you must use neutral

#### DETERGENT

Cleaning should be done by rubbing with a sponge and then rinsing thoroughly. For satin surfaces, clean following the satin finish. These instructions are necessary to avoid yellow spots that may surface due to a failure and correct cleaning before use.

#### PROTECTIVE PLASTIC

It is recommended, immediately after purchase, to remove the adhesive plastic from the surfaces that protects them, because, if leaved for a long time, it may release the residuals of glue which, not being visible immediately, may could turn yellow with the dirt deposit.

#### PERIODIC MAINTENANCE

For a good preservation of the stainless steel surfaces, especially when exposed to the atmosphere, it is sometimes advisable to resort to a periodical maintenance that consists of a washing with simple water and soap or neutral detergent taking care to provide to a good rinsing and to a subsequent careful drying. In any case it is always advisable to clean, rinse and dry the steel carefully with every use.

#### CONTACT WITH OTHER SURFACES

If other items are used above the steel surfaces, such as anti slip rubber mats, it is advisable to make sure that before placing them on the stainless steel surface, both surfaces are equally clean and dry with care. If this foresight is not implemented, in time the yellowing of stainless steel will be risked. At the end of the use, it is advisable to take care to remove the articles from the stainless steel surface when they are no longer used. The use of compounds that can release ionicloro, such as hydrochloric acid or its solutions, must be avoided, as stainless steel would be damaged.

#### THINGS TO AVOID:

- Use of wool or steel wool - vinegar vapours - hydrocyanic acid 100% - linoleic acid 100% up to 100 ° - Muriatic acid - sulphuric acid 5% boiling - sulphuric acid smoking oleum 50 ° - sulphurous acid 100% - Molten aluminium - sulphur dioxide 90% - sodium bisulfate 15% at 85 ° - saturated bleach - ferric chloride 5-50% - Ferrous chloride 10-20% - moist chlorine gas - calcium hypochlorite - sodium hypochlorite - Carbon tetrachloride - zinc chloride-ammonium phosphate - sodium phosphate - calcium hydroxide, Magnesium and potassium - borate sodium - sodium peroxide.

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### 1) SECURITY:

Important safety information

All safety messages in the main text of this manual begin with the terms WARNING, PROHIBITION and PRECAUTION, which have the following meanings:

**WARNING:** Important information to warn the user of a situation that can cause serious injury and material damage if you do not follow the instructions.

**PROHIBITION:** Important information to warn the user that the aforementioned situation is prevented for the purpose of serious injury and damage.

**PRECAUTION:** Important information to Inform the user about how to prevent material damage or avoid a situation that may cause minor injuries.

Basic elements of safety

In addition to the warnings and precautions contained in this manual, please read and observe the following safety instructions:

1. read all instructions carefully;
2. keep instructions for future references;
3. follow all warnings and precautions printed on the appliance;
4. use a damp cloth to clean the parts

**WARNING:** Never try to perform maintenance or troubleshooting tasks other than those described in this manual.

### ATTENTION:

The company LORY PROGETTI VETERINARI SRL, will not be held responsible for any inconvenience, breakage, accidents due to failure to respect or to the non-application of the indications contained in this user manual.

LORY PROGETTI VETERINARI reserves all rights to this manual, no reproduction in whole or in part is permitted without the written authorization of LORY PROGETTI VETERINARI SRL

**WARNING:** the company LORY PROGETTI VETERINARI SRL is not responsible for any damages or expenses caused by the use of components other than those provided by the manufacturer.

### 1) DESCRIPTION OF EQUIPMENT AND GENERAL OPERATING PRINCIPLES:

Manufacturer company: Lory Progetti Veterinari srl

Model: GA042

Type: CAGE FOR VETERINARY USE

Characteristics: STANDARD STAY CAGE 2 PLACES

AISI 304 stainless steel cage

### 2) INTENDED USE OF:

The GA042 cage may be used for medical purposes as a shelter for small and medium-sized animals, but weighing not more than 100 kg. The GA042 cage cannot be used outside the field of application for which it has been made, therefore, uses such as human use, as a forklift for things or materials, etc., should be avoided.

The GA042 cage may only be used for purposes for which any other use outside this scope has been prohibited.

### 3) NUMBER AND LOCATION OF JOBS:

The cage GA042 is designed to work with only one person using only the opening of doors

### 4) FIELD OF USE:

The GA042 cage is suitable for the use of animals, even of a certain complexity.

The GA042 cage is particularly suitable for all those situations in which it is important that the animal is at ease without any other animals nearby.

It is therefore advisable that the use of the cage should be allowed only to persons who are informed about the use of the cage, so that children and inexperienced people should not be allowed to use it.

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## RECOVERY CAGE

### 5) TECHNICAL DATA:

Manufacturing company: Lory Progetti Veterinari srl - ITALY

Built:

AISI 304 stainless steel cage

Supply company of spare parts Lory Progetti Veterinari srl

Maximum dimensions of space

Length 60 cm

Width 72 cm

Height 170 cm

Net weight 60 kg

### 6) PROHIBITIONS:

The company LORY PROGETTI VETERINARI SRL is not responsible for damages to persons or things caused directly or indirectly by the removal of repairs:

1. PROHIBITION: it is forbidden to remove fixed and mobile protections;
2. PROHIBITION: tampering with safety devices is prohibited;
3. PROHIBITION: it is forbidden to use the cage in potentially explosive atmospheres.

### 7) ELIMINATION OF THE RISK:

7.1) Operating hazards and warning signs

The Veterinary cage is constructed so that it does not present sharp elements, sharp edges, sharp corners and sharp edges;

### 8) THE PACKAGING SHALL:

The packaging of the veterinary cage is provided directly by the supplier company.

The cage is inserted into a sturdy cardboard box.

It is not necessary to proceed to the packing of the cage for Veterinary if some parts are wet or wet.

### 9) THE TRANSPORT SHALL:

The company LORY PROGETTI VETERINARI SRL will not be liable for damage to goods and persons for transport that do not comply with the conditions set out below.

During transport, the Veterinary cage must be protected from atmospheric agents, for the purpose the packaging if carried out in the manner above is suitable as long as it is present intact in all parts.

9.1) storage

In case the Veterinary Cage is put into storage for a certain period, it is important that this happens in a dry environment and that the casing is not subjected to stacking or crushing.

9.2) The loading and unloading

The loading, unloading and transport of the Veterinary Cage must be carried out taking care that it does not have to suffer tipping, falls and shocks.

The cage is supplied packed by the manufacturer.

Cage churning operations shall include the following sequence:

1. ensure that the carton box is positioned in such a way that the upper part of the carton is actually inserted in the upper part of the carton if it is not rotated until this condition is verified;
2. proceed with opening the box by rotating the flaps by 270°;
3. at this point, the cage shall be raised on both sides until the edge of the box is overcome and subsequently deposited on the ground;

### 10) THE INSTALLATION OF:

The room where the Veterinary cage is to be placed must be ventilated and dry.

The artificial lighting must be as high as possible on all parts of the machine, taking care not to create areas of shadow in the parts of intervention of the operator.

### 11) OPERATION:

The GA042 Veterinary Cage has a very simple operation, in fact it can be performed all operations without special learning courses and by staff not specially trained.

### 12) ASSEMBLY, DISASSEMBLY, MAINTENANCE AND REPAIR:

The assembly and disassembly operations must be carried out by personnel of the supplier or by specialized mechanics who have followed a specific preparation course at the supplier.

It is understood that for the duration of the warranty no repairs may be made to these parties by a person other than the manufacturer under penalty of immediate forfeiture of the warranty.

### 13) LUBRICATION:

The GA042 Veterinary Cage does not need any lubrication.

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## RECOVERY CAGE

### 14) SPARE PARTS:

For the parts of exchange is necessary that they are used original pieces produced from the company constructor LORY PROGETTI VETERINARI SRL

The following information shall be given for the order of parts:

Model GA042

Serial number:

For a greater speed and precision in the orders it is always better to refer to the diagram of the annexed componentistica to the present manual in which case it is sufficient to replace to the given data the following ones:

Model number

Number of pieces

### 15) ANALYSIS OF THE NOISE:

The GA042 veterinary cage is quiet.

Guarantee: part that remains attached to the user book

### 16) CERTIFICATE OF GUARANTEE:

Animal cage model for veterinary: GA042

Date of purchase:

Validity: 1 year with the date of sale to the final operator using it

The product warranty is valid provided that:

- 1) the product remains in the possession of the original buyer;
- 2) the product is used privately;
- 3) the product has not suffered damage due to accidents and/ or improper use, tampering and/ or changes by personnel not authorized by LORY PROGETTI VETERINARI SRL

In the event of a defect in the material or in the manufacture, LORY PROGETTI VETERINARI SRL will, at its choice, repair or replace. All related expenses will be borne by LORY PROGETTI VETERINARI SRL.

Instead, the buyer is responsible for the shipping costs that must be prepaid. In addition, the product must be properly packaged so as to avoid damage to the same during transport, damage that will not be recognized by the company.

Duration:

The warranty period is 12 months from the date of original purchase as evidenced by the copy of the invoice.

The procedure shall:

- 1) activation: for the validity of the guarantee the invoice or the accompanying document will be valid, to be delivered at the time of any problem
- 2) interventions: LORY PROGETTI VETERINARI SRL will check the equipment and then, if necessary, proceed to repair or replace.