



BIEFFE ITALIA s.r.l.

SUPER EASY and EASY TROLLEYS



Assembling instructions and maintenance manual



BIEFFE ITALIA s.r.l.
Via Palma 284, 80040 Striano (NA)
Tel.: +39 081 8654662 – Fax: +39 081 8654723
bieffeita@libero.it

INTRODUCTION

Dear Customer,

Your choice of our appliance means, for our Company, your particular recognition of the care of our designers, in the achievement of an ergonomic, reliable and pleasant design product.

We would thank you in advance for your choice and we are at your complete disposal for everything you need.



You are kindly requested to pay particular attention to this manual, before using the item, as it contains important recommendations on the correct use and maintenance of the product.

Have a good work!

INFORMATION

Our SUPER EASY and EASY trolleys, both BASIC and POWER versions, are made of iron bases and shelves with aluminium columns.

The chassis, with particular mechanical and chemical treatments, is made of oven epoxy powder coated aluminium, that doesn't let the structure become wasted.

The shelves have a special edge to avoid the falling of the objects.

The corners of the shelves, the plate and the buffer of the wheels are made of ABS.

The wheels, made of polyamide, with and without brake, are available in both conductive and non conductive version.

PACKAGING HANDLING

After taking away the packaging, do not leave the different elements (carton boxes, plastic bags, polystyrene, etc.) in the environment and do not leave them in children hands.

Anyway do not throw away the packaging before verifying the correct operation of the device, as in case of giving it back to the supplier, the device has to be put in its own original packaging.

All the different elements of the packaging, have to be handled following the local Directives regarding the disposal of the waste.

If it is necessary to dispose the trolley, at the end of its useful life, or for other reasons, it is important to consider that the trolley has to be disposed following the directions of the Directives that regulate the electromechanical/electrical devices, for which the same Directives require to reutilization/recycle the parts of the device, to avoid damages to the environment and consequently for the people health.

Ignoring the Directives regarding the disposal of the waste, please contact the qualified local Authorities.

KIT

SUPER EASY and EASY trolleys, in both BASIC and POWER versions, with the optionals, are usually supplied assembled or, following the customer request, in non assembled kit.

In this case, the assembling activities are very very easy, and are approximatively transferred in the following paragraph.

ASSEMBLING DIRECTIONS



Figure A – SUPER EASY BASIC

As the possible configurations are too many to be transferred in the present manual, in the figure A is shown only an example of SUPER EASY BASIC trolley already assembled, and in the figure B is shown only an example of EASY POWER already assembled.

Both the figures have to be considered only like a demonstration to the assembling activities when the trolleys are supplied in non assembled kit.



Figure B – EASY POWER

The assembling activities are very very simple, as on the columns are already placed, in suggested but adjustable position, the screws for fixing the different components.

So, we suggest the following operations:

A) SUPER EASY BASIC trolley: under the column are the M10x40 screws, useful to fix it on the basis, where are already made the suitable holes.

Unscrew the screws, then fix the column over the basis, screw again the screws and tighten them, with the suitable hexagonal spanner supplied in the assembling kit.

To fix a shelf unscrew the M6x10 screws that are already placed, in suggested but adjustable position, then place the shelf itself with its holes opposing the grooves of the column, screw again the screws and tighten them, with the suitable hexagonal spanner supplied in the assembling kit.

Please follow the same directions to fix other shelves or optionals.

B) EASY POWER trolley: to fix the columns to the basis, follow the above directions considering that the trolley has 2 columns for the shelves plus 2 power columns.

As regards the shelves, they have to be fixed both to the columns for the shelves themselves and to the power columns, when are already placed the screws M6x10.

Please follow the same directions to fix optionals.

To assemble the wheels, fix them to the pivot under the basis, tighten them with a big pressure.

USE PURPOSE

SUPER EASY and EASY trolleys, in both BASIC and POWER versions are suitable to support, to move, or to connect, medical equipment, in outpatient's departments, and private medical ambulatories.

These trolleys must only be used for the purpose for which they were designed.

All other uses must be considered dangerous.

GENERAL WARNING

After removing the packaging, check that the product has not been damaged in transit.

In case of doubt do not use the appliance and apply to the supplier.

These advices must only be used for the purpose for which they were designed.

If the trolley is in power version, its use, like any electrical appliance involves the observance of certain basic rules, in particular:

- do not touch the appliance with damp hands
- do not use the appliance while bare-foot
- do not pull the plugb from power socket by the power cord of the appliance itself
- do not expose the appliance to adverse elements (sun, rain, etc.)
- do not let the children use the advice.

The manufacturer can't be considered responsible for damages caused by an incorrect use of the product.

SPECIAL WARNINGS FOR TROLLEY IN POWER VERSION

In case of doubt do not use the appliance and apply to the supplier.

The electrical safety of this appliance is only guaranteed when it is correctly installed and connected to the earthed system, according to the latest issue of the Regulations Conformity with these requirements.

So it is necessary verify the safety requirements and, in case of doubt, ask a check of the earthed system from a suitable qualified person.

Besides it is important to respect the following recommendations:

- check that the current carrying capacity of the wirings and socket are suitable for the maximum power of the device as shown on the label. In case of doubt, seek advice from a suitable qualified person
- to avoid dangerous overheating, it is advisable to fully unroll the power cord
- do not leave the appliance plugged in when unnecessarily.

If power cord becomes damaged, replace it promptly with the power cord: EN 60 320-C15 - **outlet** CEI 23-16/VII (SCHUKO CEE 7-S.S. VII).

The manufacturer can't be considered responsible for any damage caused by an incorrect installation and earthing of the appliance.

MAINTENANCE

It is necessary undergo safety check the trolleys, if in power version, once a year, from a suitable qualified person.

CLEANING DIRECTIONS

Clean the appliance with water or with non corroding substances.

If the trolley is in power version, disconnect it before cleaning operations.

Do not use any solvent to clean the appliance.

GUARANTEE

The supplier warrants that the appliance is safe to use and it will have no harmful or injurious effects if used in accordance with the supplier's directions. The supplier further warrants that the appliance is free from defects in materials and workmanship, his liability under this warranty being limited to replace, as soon as possible, all such parts which are proven defective.

The guarantee of the device is of 24 months, from the date of the purchase.

This guarantee is valid only if the appliance is used for the purpose for which it is manufactured and sold.

This guarantee is valid 24 months, even if the trolley has never been used.

Parts acquired from other manufacturers have the same guarantee that is given to the supplier by the manufacturers.

Shipping cost for repairing actions are paid by the Customer, even if the appliance is under warranty.

The repairing actions do not modify the expiry date of guarantee.



DECLARATION OF CONFORMITY



Directives 2007/47/CE, 2006/95/CE, 2004/108/CE

BIEFFE ITALIA s.r.l.

declare under our responsibility that the product

**SUPER EASY and EASY
TROLLEY
in basic/power version**

**is in conformity with the following standards or
other normative document**

EC Directive 2007/47 medical device of Class I

EC Directive 2006/95

EC Directive 2004/108

Striano (NA), March 2014

BIEFFE ITALIA s.r.l.
Dr. Flavio Ferrazzano