VASCA PRE-OPERATORIA

PRE-SURGICAL TUB

VA006/DX - VASCA PREOPERATORIA MOD. "ARNICA"

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Dimensioni totali: L130x62xH93 cm

PRE-OPERATORY TUB TABLE "ARNICA"

Made of type Aisi 304 stainless steel, with seamless bowl for total disinfection. Fitted with pull-out spray mixer. Frame unit made of epoxy-coated steel.

Cabinet dimensions: W100xD62x H67 cm

Inner tub dim.: W116xD50x Bowl depth: 20 cm

Total dimensions: W153xD62xH93 cm

VA006/130/S X - PRE-OPERATORY TUB TABLE "ARNICA"

(Connection for mixer on the right). Small version of the same model. Cabinet dimensions: W100xD62x H67 cm. (Attacco per miscelatore a sinistra) (Connection for mixer on the left)

Inner tub dimensions: W96xD50x Bowl depth: 20 cm

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The undersigned Lory Progetti Veterinarians srl, established in Via B. Buozzi 3/A, Cavriago - REGGIO EMILIA **DECLARE** that the equipment of which follows the characteristics: Denomination: **VA006**

The appliance is built with materials corresponding to the European regulations for medical use in the veterinary field. The instrument does not contain any electrical parts and has a manual operation.

Complies with the following directives: 2001/95/CE (general product safety) 2014/30/UE (Electromagnetic compatibility)

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TREATMENT INSTRUCTIONS FOR AISI 304 STAINLESS STEEL

FIRST CLEANING

Before using equipment made of aisi 304 stainless steel, it's recommended to clean the entire internal and external surface to remove any possible working residual; according to the type of equipment,

FOLLOW THIS INDICATIONS

- On a surface that will come into direct contact with water you must use a no-aggressive or anti-calcare product. (Es like viacal)
- On a surface that will not come everytime into direct contact with water you must use neutral

DETERGENT

Cleaning should be done by rubbing with a sponge and then rinsing thoroughly. For satin surfaces, clean following the satin finish. These instructions are necessary to avoid yellow spots that may surface due to a failure and correct cleaning before use.

PROTECTIVE PLASTIC

It is recommended, immediately after purchase, to remove the adhesive plastic from the surfaces that protects them, because, if leaved for a long time, it may release the residuals of glue which, not being visible immediately, may could turn yellow with the dirt deposit.

PERIODIC MAINTENANCE

For a good preservation of the stainless steel surfaces, especially when exposed to the atmosphere, it is sometimes advisable to resort to a periodical maintenance that consists of a washing with simple water and soap or neutral detergent taking care to provide to a good rinsing and to a subsequent careful drying. In any case it is always advisable to clean, rinse and dry the steel carefully with every use.

CONTACT WITH OTHER SURFACES

If other items are used above the steel surfaces, such as anti slip rubber mats, it is advisable to make sure that before placing them on the stainless steel surface, both surfaces are equally clean and dry with care. If this foresight is not implemented, in time the yellowing of stainless steel will be risked. At the end of the use, it is advisable to take care to remove the articles from the stainless steel surface when they are no longer used. The use of compounds that can release ionicloro, such as hydrochloric acid or its solutions, must be avoided, as stainless steel would be damaged.

THINGS TO AVOID:

- Use of wool or steel wool - vinegar vapours - hydrocyanic acid 100% - linoleic acid 100% up to 100° - Muriatic acid - sulphuric acid 5% boiling - sulphuric acid smoking oleum 50° - sulphurous acid 100% - Molten aluminium - sulphur dioxide 90% - sodium bisulfate 15% at 85° - saturated bleach - ferric chloride 5-50% - Ferrous chloride 10-20% - moist chlorine gas - calcium hypochlorite - sodium hypochlorite - Carbon tetrachloride - zinc chloride-ammonium phosphate - sodium phosphate - calcium hydroxide, Magnesium and potassium - borate sodium - sodium peroxide.

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THE INSTRUCTION BOOKLET VA006

1) SECURITY:

Important Information About Security

All safety messages in the main text of this manual begin with the terms WARNING, PROHIBITION and PRECAUTION, which have the following meanings:

WARNING: Important information to warn the user of a situation that can cause serious injury and material damage if you do not follow the instructions.

BAN: Important information to warn the user that the aforementioned situation is prevented for the purpose of serious injury and damage.

PRECAUTION: Important information to Inform the user about how to prevent material damage or avoid a situation that may cause minor injuries.

Basic elements of safety

In addition to the warnings and precautions contained in this manual, please read and observe the following safety instructions:

1. read all instructions carefully;
2. keep instructions for future references;
3. follow all warnings and precautions printed on the appliance;
4. use a damp cloth for cleaning.

WARNING: non never try to perform maintenance work or troubleshooting other than those described in this manual.

Disconnect the appliance from the wall socket and entrust the technical assistance work to qualified personnel in the following cases:

1. sand the product does not function normally when following the instructions for use and having carried out the checks described in the manual, because inadequate adjustment of other parameters may cause damage and require many hours of work by a qualified technician to restore normal operation;
5. the equipment has fallen
6. if the performance of the apparatus has changed, indicating the need for technical assistance.

ATTENTION:

The company LORY PROGETTI VETERINARI SRL, will not be held responsible for any inconvenience, breakage, accidents due to failure to respect or to the non-application of the indications contained in this user manual.

LORY PROGETTI VETERINARI reserves all rights to this manual, no reproduction in whole or in part is permitted without the written authorization of LORY PROGETTI VETERINARI SRL

WARNING: the company LORY PROGETTI VETERINARI SRL is not responsible for any damages or expenses caused by the use of components other than those provided by the manufacturer.

WARNING: Although it is not necessary, it is considered appropriate for the equipment to be relieved of tension after each shift, and in particular at the end of the shift.

1) DESCRIPTION OF EQUIPMENT AND GENERAL OPERATING PRINCIPLES:

Manufacturer: Lory Progetti Veterinari srl

Model: VA0086

Type: PRE SURGICAL TANK

Features removable mixer tap

Made entirely of 304 stainless steel.

Company supplier of spare parts Lory Progetti Veterinari sr

2) INTENDED USE:

The VA006 pre surgical tank can be used for medical use.

The VA006 pre surgical tank can not be used outside the field of application for which it was made, are therefore to be avoided uses such as: human use, such as elevator for things or materials, etc.

The VA006 pre surgical tank can be equipped with specific accessories provided only and exclusively by the manufacturer.

The VA006 pre surgical tank can be used exclusively for the use for which it has been made any other use outside this area is prohibited.

3) NUMBER AND LOCATION OF JOBS:

The VA006 pre surgical tank is designed to work with a single or double person by lifting the removable mixer.

4) FIELD OF USE:

The VA006 pre surgical tank is particularly suitable for the surgical room

The use of the VA006 pre surgical tank noted the simplicity of pre-operation and the very low speed of maneuvering, can be performed by personnel not particularly trained, but at least read these instructions for use.

It is therefore advisable that the use of the tank, is allowed only to persons informed about the use therefore is not to allow the use to children and the inexperienced'

VASCA PRE-OPERATORIA

PRE-SURGICAL TUB

5) TECHNICAL DATA:

Manufacturer: Lory Progetti Veterinari srl
 Made of stainless steel, working with extractable mixer
 Maximum dimensions of space
 Length 130/153 cm
 Width 62 cm
 Height 93 cm
 Net weight 30 kg

6) PROHIBITION OF:

The company LORY PROGETTI VETERINARI SRL is not responsible for damage to persons or property caused directly or indirectly by the removal of repairs or the deactivation of safety devices:

1. PROHIBITION: it is forbidden to remove fixed and mobile protections;
2. PROHIBITION: tampering with safety devices is prohibited;
3. PROHIBITION: it is forbidden to carry out on moving organs any repair or registration operation;
4. PROHIBITION: it is forbidden to use the sink in potentially explosive atmospheres

7) ELIMINATION OF RISK:

7.1) Operational risks and monitoring cartels

The pre surgical tank is constructed so that it does not present sharp elements, sharp edges, sharp corners and sharp edges;

8) THE PACKAGING:

The packaging of the pre surgical tank is supplied directly by the supplier.

The pre surgical tank is normally covered by a polyethylene film stretched to cover all the parts of the tank.

Then the tank so packed is inserted inside a sturdy cardboard box.

Before proceeding with the laying of the polyethylene sheet will be covered with foam all the protruding parts that can compromise the integrity of the cloth.

It is not necessary to proceed to the packaging of the pre surgical tank if some parts are wet or wet, this could cause rapid and important oxidation phenomena of the metal parts and mechanical and electrical organs not in stainless steel.

9) THE TRANSPORT:

The company LORY PROGETTI VETERINARI SRL will not be liable for damage to goods and persons for transport that do not comply with the conditions set out below.

During transport, the pre surgical tank must be protected from the elements, for this purpose the packaging if done with the above methods is suitable as long as it is present intact in all parts.

9.1) The storage

In case the pre surgical tank is put in stock for a certain period, it is important that this happens in a dry environment and that the envelope is not subjected to stacking or crushing.

9.2) Loading and unloading

The loading, unloading and transport of the pre surgical tank must be carried out taking care that it is not subject to tipping, falls and shocks.

the pre surgical tank is supplied packed by the manufacturer.

the pre surgical tank shall be chucked in the following sequence:

1. ensure that the carton box is positioned in such a way that the upper part of the carton box is actually inserted in the upper part of the carton if it is not rotated until this condition is verified;
2. open the box by rotating the flaps by 270°;
3. at this point, proceed with lifting the pre surgical tank on both sides until the edge of the box is overcome and then is deposited on the ground;
4. the polyethylene film shall then be removed.

10) THE INSTALLATION:

The room where the pre surgical tank should be installed, must be ventilated and dry.

The artificial lighting must be as high as possible on all parts of the machine, taking care not to create areas of shadow in the parts of intervention of the operator.

The work surface must be well lit with an illumination of at least 400 Lux at a height of 1,00 m from the floor.

The minimum dimensions of the room to install the pre surgical tank are:

length: 3,20 m.

width: 2,60 m.

width of service door compartment: 0,70 m.

11) OPERATION OF:

The VA006 pre surgical tank has a very simple operation, in fact all operations can be performed without special learning courses and by personnel not specially trained.

VASCA PRE-OPERATORIA

PRE-SURGICAL TUB

12) ASSEMBLY, DISASSEMBLY, MAINTENANCE AND REPAIR:

The assembly and disassembly operations must be carried out by personnel of the supplier or by specialized mechanics who have followed a specific preparation course at the supplier.

With regard to repair operations:

maintenance of mechanical parts

Mechanical parts shall be repaired, assembled and disassembled by personnel with specific training in carpentry, mechanics or adjustment;

13) LUBRICATION OF:

The VA006 pre surgical tank does not require any lubrication.

14) SPARE PARTS:

For the parts of exchange it is necessary that they are used original pieces produced from the company constructor LORY PROGETTI VETERINARI SRL

The following information shall be given for the order of parts:

Model of the machine VA006

Serial number and year of manufacture of the machine MATRICOLA ...

Number and description of/i piece/s required/s

For a greater speed and precision in the orders it is always better to refer to the diagram of the annexed componentistica to the present manual in which case it is sufficient to replace to the given data the following ones:

Model number

Position number

Number of pieces

15) ANALYSIS OF NOISE:

The VA006 pre surgical tank is particularly quiet

Guarantee: part that remains attached to the user book

16) GUARANTEE CERTIFICATE:

Surgical sink model: VA006

The number:

Date of purchase:

Validity: 1 year with the date of sale to the final operator using it

The product warranty is valid provided that:

1) the product remains in the possession of the original buyer;

2) the product is used privately;

3) the product has not suffered damage due to accidents and/ or improper use, tampering and/ or changes by personnel not authorized by LORY PROGETTI VETERINARIANS SRL