ACCESSORI VASCHE

TUB ACCESSORIES

GOC003

GOCCIOLATOIO

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SPLASH PROTECTION

It allows to obtain maximum hygiene protecting wall from splashes and hairs during the washing. Made of Aisi 304 stainless steel with S/B finish. Open on the right or left side on request and need.

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The undersigned Lory Progetti Veterinarians srl, established in Via B. Buozzi 3/A, Cavriago, RE DECLARE That the equipment of which follows the characteristics: Denomination: GOC003

THIS ARTICLE DOES NOT REQUIRE INSTRUCTIONS BOOKLET

The appliance is built with materials corresponding to the European regulations for medical use in the veterinary field. The instrument does not contain any electrical parts and has a manual operation. Complies with the following directives: 2001/95/CE (general product safety) 2004/108/EC (Electromagnetic compatibility).

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TREATMENT INSTRUCTIONS FOR AISI 304 STAINLESS STEEL

FIRST CLEANING

Before using equipment made of aisi 304 stainless steel, it's recommended to clean the entire internal and external surface to remove any possible working residual; according to the type of equipment,

FOLLOW THIS INDICATIONS

- On a surface that will come into direct contact with water you must use a no-aggressive or anti-calcaree product. (Es like vialca)
- On a surface that will not come everytime into direct contact with water you must use neutral

DETERGENT

Cleaning should be done by rubbing with a sponge and then rinsing thoroughly. For satin surfaces, clean following the satin finish. These instructions are necessary to avoid yellow spots that may surface due to a failure and correct cleaning before use.

PROTECTIVE PLASTIC

It is recommended, immediately after purchase, to remove the adhesive plastic from the surfaces that protects them, because, if leaved for a long time, it may release the residuals of glue which, not being visible immediately, may could turn yellow with the dirt deposit.

PERIODIC MAINTENANCE

For a good preservation of the stainless steel surfaces, especially when exposed to the atmosphere, it is sometimes advisable to resort to a periodical maintenance that consists of a washing with simple water and soap or neutral detergent taking care to provide to a good rinsing and to a subsequent careful drying. In any case it is always advisable to clean, rinse and dry the steel carefully with every use.

CONTACT WITH OTHER SURFACES

If other items are used above the steel surfaces, such as anti slip rubber mats, it is advisable to make sure that before placing them on the stainless steel surface, both surfaces are equally clean and dry with care. If this foresight is not implemented, in time the yellowing of stainless steel will be risked. At the end of the use, it is advisable to take care to remove the articles from the stainless steel surface when they are no longer used. The use of compounds that can release ionicloro, such as hydrochloric acid or its solutions, must be avoided, as stainless steel would be damaged.

THINGS TO AVOID:

- Use of wool or steel wool - vinegar vapours - hydrocyanic acid 100% - linoleic acid 100% up to 100 ° - Muriatic acid - sulphuric acid 5% boiling - sulphuric acid smoking oleum 50 ° - sulphurous acid 100% - Molten aluminium - sulphur dioxide 90% - sodium bisulfate 15% at 85 ° - saturated bleach - ferric chloride 5-50% - Ferrous chloride 10-20% - moist chlorine gas - calcium hypochlorite - sodium hypochlorite - Carbon tetrachloride - zinc chloride-ammonium phosphate - sodium phosphate - calcium hydroxide, Magnesium and potassium - borate sodium - sodium peroxide.

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1) SECURITY:

Important safety information

All safety messages in the main text of this manual begin with the terms WARNING, PROHIBITION and PRECAUTION, which have the following meanings:

WARNING: Important information to warn the user of a situation that can cause serious injury and material damage if you do not follow the instructions.

PROHIBITION: Important information to warn the user that the aforementioned situation is prevented for the purpose of serious injury and damage.

PRECAUTION: Important information to Inform the user about how to prevent material damage or avoid a situation that may cause minor injuries.

Basic elements of safety

In addition to the warnings and precautions contained in this manual, please read and observe the following safety instructions:

1. read all instructions carefully;
2. keep instructions for future references;
3. follow all warnings and precautions printed on the appliance;
4. use a damp cloth to clean the parts

WARNING: Never try to perform maintenance or troubleshooting tasks other than those described in this manual.

ATTENTION:

The company LORY PROGETTI VETERINARI SRL, will not be held responsible for any inconvenience, breakage, accidents due to failure to respect or to the non-application of the indications contained in this user manual.

LORY PROGETTI VETERINARI reserves all rights to this manual, no reproduction in whole or in part is permitted without the written authorization of LORY PROGETTI VETERINARI SRL

WARNING: the company LORY PROGETTI VETERINARI SRL is not responsible for any damages or expenses caused by the use of components other than those provided by the manufacturer.

1) DESCRIPTION OF EQUIPMENT AND GENERAL OPERATING PRINCIPLES:

Manufacturer company: Lory Progetti Veterinari srl

Model: GOC003

Type: ACCESSORIES FOR GROOMING

Characteristics: SPLASH PROTECTION

AISI 304 stainless steel cage

2) INTENDED USE OF:

The splash protection GOC003 can not be used outside the scope for which it was made, are therefore to be avoided uses such as: human use, as a forklift for things or materials, etc.

The splash protection GOC003 can be used only for the use for which it has been realized any other use outside this area is forbidden.

3) NUMBER AND LOCATION OF JOBS:

The splash protection GOC003 is designed to work with just one person.

It is absolutely forbidden to place feet/hands or equipment on the basis of the splash protection itself.

4) FIELD OF USE:

splash protection GOC003 is particularly suitable to be attached to the toilet tub

The use of the GOC003 ascertained the simplicity of operation and the very low speed of maneuver, can be carried out by personnel not particularly trained, but that at least has read the present instructions for the use.

5) TECHNICAL DATA:

Manufacturer: Lory Progetti Veterinari srl

Built:

top made of stainless steel

Maximum dimensions of space

length 146 cm

width 50 cm

ACCESSORI VASCHE

TUB ACCESSORIES

6) PROHIBITIONS:

The company LORY PROGETTI VETERINARI SRL is not responsible for damages to persons or things caused directly or indirectly by the removal of repairs:

1. PROHIBITION: it is forbidden to remove fixed and mobile protections;
2. PROHIBITION: tampering with safety devices is prohibited;
3. PROHIBITION: it is forbidden to use the splash protection in potentially explosive atmospheres.

7) ELIMINATION OF THE RISK:

7.1) Operating hazards and warning signs

The splash protection is constructed so that it does not present sharp elements, sharp edges, sharp corners and sharp edges;

8) THE PACKAGING SHALL:

The packaging of the splash protection is provided directly by the supplier company.

The splash protection is normally covered by a polyethylene film spread over all parts.

It is then placed inside a sturdy cardboard box.

Before proceeding to the drafting of the polyethylene sheet, all protruding parts that may compromise the integrity of the sheet will be covered with foam.

It is not necessary to proceed to the packaging of the splash protection if some parts are wet or wet, this could cause rapid and important oxidation phenomena of metal parts and mechanical and electrical parts not in stainless steel.

9) THE TRANSPORT:

The company LORY PROGETTI VETERINARI SRL will not be liable for damage to goods and persons for transport that do not comply with the conditions set out below.

During transport, the splash protection must be protected from atmospheric agents, for the purpose the packaging if carried out in the manner above is suitable as long as it is present intact in all parts.

9.1) storage

In case the splash protection is put into storage for a certain period, it is important that this happens in a dry environment and that the casing is not subjected to stacking or crushing.

9.2) The loading and unloading

The loading, unloading and transport of the splash protection must be carried out taking care that it does not have to suffer tipping, falls and shocks.

The splash protection is supplied packed by the manufacturer.

The following sequence shall be used for the buffing of the splash protection:

1. ensure that the carton box is positioned in such a way that the upper part of the carton box is actually inserted in the upper part of the carton if it is not rotated until this condition is verified;
2. proceed to open the box by rotating the flaps by 270°;
3. at this point, the tank is lifted on both sides until the edge of the box is overcome and then deposited on the ground;
4. The polyethylene film shall then be removed.

10) THE INSTALLATION:

The room where the splash protection is to be installed must be ventilated and dry.

The artificial lighting must be as high as possible on all parts of the machine, taking care not to create areas of shadow in the parts of intervention of the operator.

11) OPERATION:

The splash protection GOC003 has a very simple operation, in fact can be performed all operations without special courses and by staff not specially trained.

12) ASSEMBLY, DISASSEMBLY, MAINTENANCE AND REPAIR:

The assembly and disassembly operations must be carried out by personnel of the supplier or by specialized mechanics who have followed a specific preparation course at the supplier.

It is understood that for the duration of the warranty no repairs may be made to these parties by a person other than the manufacturer under penalty of immediate forfeiture of the warranty.

13) LUBRICATION:

The splash protection GOC003 does not need any lubrication.

ACCESSORI VASCHE

TUB ACCESSORIES

14) SPARE PARTS :

For the parts of exchange is necessary that they are used original pieces produced from the company constructor LORY PROGETTI VETERINARI SRL

The following information shall be given for the order of parts:

Model GOC003

Serial number and year of manufacture of the machine MATRICOLA ...

Number and description of/i piece/s required/s

For a greater speed and precision in the orders it is always better to refer to the diagram of the annexed componentistica to the present manual in which case it is sufficient to replace to the given data the following ones:

Model number

Position number

Number of pieces

15) ANALYSIS OF THE NOISE:

The splash protection GOC003 is silent.

Guarantee: part that remains attached to the user book

16) CERTIFICATE OF GUARANTEE:

Animal grooming bath model: GOC003

New Year. freshman :

Date of purchase:

The product warranty is valid provided that:

- 1) the product remains in the possession of the original buyer;
- 2) the product is used privately;
- 3) the product has not suffered damage due to accidents and/ or improper use, tampering and/ or changes by personnel not authorized by LORY PROGETTI VETERINARI SRL

In the event of a defect in the material or in the manufacture, LORY PROGETTI VETERINARI SRL will, at its choice, repair or replace. All related expenses will be borne by LORY PROGETTI VETERINARI SRL.

Instead, the buyer is responsible for the shipping costs that must be prepaid. In addition, the product must be properly packaged so as to avoid damage to the same during transport, damage that will not be recognized by the company.

Duration:

The warranty period is 12 months from the date of original purchase as evidenced by the copy of the invoice.

The procedure shall:

- 1) activation: for the validity of the guarantee the invoice or the accompanying document will be valid, to be delivered at the time of any problem
- 2) interventions: LORY PROGETTI VETERINARI SRL will check the equipment and then, if necessary, proceed to repair or replace.